

# Compatibility Between Patient Data Manager and iPlan

## Compatibility Between Patient Data Manager and iPlan

Older versions of **Patient Data Manager** (prior to version 2.6) are not compatible with specific versions of **iPlan** and must not be used in combination because object shifts applied in **iPlan** are not reflected when exported into DICOM.

For details, refer to Field Safety Notice/Product Notification of October 08, 2019 (Brainlab Identifier CAPA-20190904-002337).

## Affected Combinations

If you want to use any of the listed **Patient Data Manager** versions with any of the listed **iPlan** versions, contact Brainlab support to request the corresponding **Patient Data Manager** software update.

Software Versions	
<b>Patient Data Manager</b>	2.1.1, 2.2.0, 2.2.1, 2.3.0, 2.3.1, 2.4.0, 2.5.0, 2.5.1
<b>iPlan CMF</b>	3.0.5, 3.0.6
<b>iPlan Cranial</b>	3.0.5, 3.0.6
<b>iPlan ENT</b>	3.0.5, 3.0.6
<b>iPlan Spine</b>	3.0.5, 3.0.6

## Explanation

If both devices are available at your facility, but it is ensured that they are not and will not be used in combination (e.g., a listed **Patient Data Manager** version is available only in the RT department, and a listed **iPlan** version is available only in a non-RT department), a **Patient Data Manager** software update is not required.

